

WET FIELD BIPOLAR COAGULATOR

SD-225 Speedway, Delhi (INDIA)



SPEEDWAY

since 1934

SPEEDWAY - Wet-Field BIPOLAR COAGULATOR is a mini Diathermy SOLID STATE for controlling bleeding during surgical procedures eliminating tissue trauma using Bipolar standard Forceps and Erasers. Tissue may be coagulated under Irrigation (Wet-Field) with Saline or in a bloody coagulating only the bleeder or particular tissue.

FEATURES:

- 1 Power required for Wet-field is 230 Volts, 50 cycle A.C \pm 10%
- 2 Foot Control system gives convenience to Surgeons.
- 3 Forceps, Erasers, cable are autoclave
- 4 Digital Display for easy setting.
- 4 Imported Forceps and Erasers can be attached.
- 6 Power control is continuously variable between 1-9.
- 7 Audible tone lets surgeon know the unit is activated.
- 8 Compact in size, Elegant Look and easily portable.

BENEFIT:


- 1 Solid state circuit system ensures safest Procedures to delicate tissue involved for Cataract, Retinal and IOL Surgery .
- 2 Can be used with irrigation (Wet-Field) or in the boldy field itself, to locate the bleeder easier to enable the Surgeon to have the better visualaization.
- 3 Useful for Ophthalmic Surgical procedures and for delicate surgery reducing TISSUE TRAUMA.
- 4 Coagulating with saline irrigation reduces tissue heating, shrinkage, drying and sticking of coagulam to the forceps tips

FAULT DIAGNOSIS:

- 1 If Unit is mot functioning check up the power supply and fuse.
- 2 While using forceps use glove to avoid grounding.
- 3 Sticking of coagulam to the forceps tips or carbon deposit at the tip-simply clean it with knife and surgical spirit.
- 4 Use spare cables or forceps for fault in cable or forceps.
- 5 Avoid direct contact of forceps tips, there should be gap for tissue only.
- 6 Unit may not work if there is any fault in cable or forceps, use spare cables or forceps.



Specifications of equipment/s are subject to change without prior notification

ISO 9001:2008;
ISO 13485:2003 



SPEEDWAY
since 1934

Speedway
SURGICAL CO.

Precision is our vision

E-mail: sales@speedwaydelhi.com Website: www.speedwaydelhi.com

+91-11-2386 6379 / 4563 0593 / 2722 7545 / 2741 7545 / 4237 2101